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Instruction for Use

Stone Grasper has 3 wires.

Indications: intended for stone manipulation and removal in the urinary track. This made out of Medical Grade Stainless Steel (304). Intended for one time use.

Contraindications: Infections, obstruction, incontinence

Potential Complications and risks: Potential damages of urethra, perforation and bleeding.

Cautions: This product should be used by only by physicians experienced in the use of such device.

Ethylene oxide sterilized. Non-pyrogenic. Check the package. if package damaged or opened do not use the products. Check the expiry date and the gauge. Do not use over expired products. Store in dry place, between 5 – 30 °C temperature, protect from sun light. Possible allergic reactions should be considered. For single Patient Use Only. Do not attempt to clean or resterilize this product. Disposal as a biohazard material after use

Procedure:

- Be sure the stone grasper in closed position and introduce it to urethroscope channel.
- Advance the stone grasper till the stone site and open the grasper over stone with pushing forward to handle.
- Pull back the handle to close the grasper. Be sure that hold the stone firmly.
- Pull back the stone grasper through urethroscope carefully while keeping firmly the stone grasper in closed position.

Note: These instructions are NOT meant to define or suggest any medical or surgical technique. The individual practitioner is responsible for the proper procedure and techniques to be used with this device.


1984 : CE mark and NB identification number

GEOTEK MEDİKAL VE SAĞLIK HİZ.LTD.ŞTİ
İVEDİK OSB AĞAÇ-METAL SİT. 1436 SK NO:12
YENİMAHALLE -ANKARA
TEL:+90312 3959852
FAX:+90312 3959853

Single Use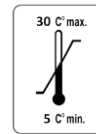

Storage limits

Expiry Date

STERILE EO

Ethylene oxide sterilized