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Instruction for Use

ENG

Indications: provides ureteral dilation and a continous working channel for the introduction of endoscopes and instrument during ureteral access procedures. The sheath protect the ureter, delicate instrument and the smaller flexible scopes during repeated instrument exchange reducing the potential of trauma

used in conjunction with the amplatz type dilators and / or balloon dilators during renal dilatation to provide non – traumatic working tract after removal of the dilator.

Contraindications: Infections, obstruction, incontinence

Potential Complications and risks: Potential damages of perforation and bleeding.

Cautions: This Kit should be used by only by physicians experienced in the use of such device.

Ethylen oxide sterilized. Non-pyrogenic. Check the package. if package damaged or opened do not use the products. Check the expiry date and the gauge. Do not use over expired products. Store in dry place, between 5 – 30 °C temperature, protect from sun light. Possible allergic reactions should be considered. For single Patient Use Only. Do not attempt to clean or resterilize this product. Disposal as a biohazard material after use

Procedure:

1. after dilating the tract with chosen size dilator, advance the access sheath with the dilator is slow but firm twisting motion, till it enters into the renal pelvis.
2. once the position is confirmed via fluoroscopy remove the dilator leaving the guidewire in place,
3. Start using sheath as working channel.

Note: These instructions are NOT meant to define or suggest any medical or surgical technique. The individual practitioner is responsible for the proper procedure and techniques to be used with this device.

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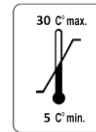
STERILE EO

Single Use Storage limits Expiry Date Ethylen oxide sterilized

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GEOTEK MEDİKAL VE SAĞLIK HİZ.LTD.STİ