<b>Doküman No: IFU-SDJ</b> Documentation No: IFU-SDJ	<b>İlk yayın tarihi: 02/05/2012</b> Valid from: 02/05/2012	<b>Revizyon no: 03</b> Revision no: 03	<b>Sayfa:1</b> Page:1
		<b>Revizyon tarihi: 09.09.2016</b> Revision date:09.09.2016	

**GEOTEK® URETERAL STENT (SINGLE J )****Kullanma Kılavuzu****TR**

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**GEOTEK® URETERAL STENT (SINGLE J )****Instruction for Use****ENG**

Ureteral Stent Kit with fully radiopaque single J stent , dilators , PTFE Guide wire and patient record card. Stents made of Tecoflex PU. Both end Open and One end Close stent types are indicated at the labels.

**Indications:** Ureteral stent is used for temporary external drainage from the ureteropelvic junction or the bladder

**Contraindications:** Serious infections, obstruction, incontinence

**Side Effects:** Kidney, bladder irritation caused by foreignbody, pain, infection, increased miction-frequency, hematuria

**Potential Complications and risks:** Disfunction caused by Dislocation, Obstruction and Encrustation of stents. It may cause calculi in the kidney.

**Cautions:** **Maximum indwelling time one month.** This Kit should be used by only by physicians experienced in the use of such device.

Regular monitoring of location and free passage of stent is necessary to evaluate stent efficiency. Inflammations of Urinary tract may increase the rate of encrustation and obstruction of the stent.

Ethylene oxide sterilized. Non-pyrogenic. Check the package. if package damaged or opened do not use the products. Check the expiry date and the gauge. Do not use over expired products. Store in dry place, between 5 – 30 °C temperature, protect from sun light. Possible allergic reactions should be considered. **For single Patient Use Only.** Do not attempt to clean or resterilize this product. Disposal as a biohazard material after use.

**Placement of Single J Stent:**

- Place the guide wire in to Renal pelvis under x-ray control.
- Insert the Stent over guide wire and advance it with help of the pusher to placement area.
- When the stents reach target area, withdraw the guide wire while hold the pusher in current position.
- After stent positioned remove the pusher gently.

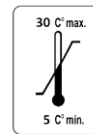
**Note:** These instructions are NOT meant to define or suggest any medical or surgical technique. The individual practitioner is responsible for the proper procedure and techniques to be used with this device.

1984 : CE mark and NB identification number

Single Use

Storage limits

Expiry Date

Ethylene oxide sterilized

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GEOTEK MEDİKAL VE SAĞLIK HİZ.LTD.ŞTİ