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### INSTRUCTIONS FOR USE:

ENG

**Indications For Use:** It is used with Automatic Biopsy Guns to harvest histological core samples from soft tissue and organs.

**Contraindications:** This needle should be used by a physician trained in interventional techniques and familiar with the possible side effects, typical findings, limitations, indications and contraindications of biopsy procedures. Physician judgment is required when considering biopsy on patients with bleeding disorder, or receiving anti-coagulant medications.

**Possible Side Effects:** Pain, fever, bleeding, hematuria and infection.

**Caution:** Ethylen oxide sterilized. Non-pyrogenic. Check the package. if package damaged or opened do not use the products. Check the expiry date and the gauge. Store in dry place, between 5 – 30 °C temperature, protect from sun light. Possible allergic reactions should be considered. It must be used only in hospitals by physicians who familiar with the device and procedures. For single Patient Use Only. Do not attempt to clean or resterilize this product. Do not use the device under MRI. After use, this product may be a potential biohazard. Handle in a manner which will prevent accidental puncture. Dispose in accordance with applicable laws and regulations.

Please double check for device function and possible damage for correct implantation and do not use if package is opened or damaged.

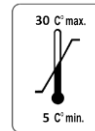
### BIOPSY PROCEDURE:

1. Identify the biopsy site preferably under US or CT.
2. Using the sterile technique, prepare the skin with antiseptic and drape.
3. If necessary, infiltrate the selected area with local anaesthetic.
4. Open the needle pouch and remove protective sheath from the needle.
5. Set the needle into the biopsy gun.
6. Follow the instructions provided by the biopsy gun manufacturer.

**Note:** These instructions are NOT meant to define or suggest any medical or surgical technique. The individual practitioner is responsible for the proper procedure and techniques to be used with this device.

Single Use

Storage limits

Expiry Date

STERILE EO

: Ethylen oxide sterilized

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GEOTEK MEDİKAL VE SAĞLIK HİZ.LTD.ŞTİ