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INSTRUCTIONS FOR USE:

ENG

Kit consists of: Biopsy Needle Guide, Automatic Biopsy Needle, Chiba Needle and sample bottles

Indications For Use: Used for USG guided Transrectal Prostate Biopsy

Contraindications: Biopsy / aspiration sets or needles should be used by a physician trained in interventional techniques and familiar with the possible side effects, typical findings, limitations, indications and contraindications of biopsy procedures. Physician judgement is required when considering biopsy on patients with bleeding disorder, or receiving anti-coagulant medications.

Possible Side Effects: Pain, fever, bleeding, hematuria and infection.

Caution: Ethylen oxide sterilized. Non-pyrogenic. Check the package. if package damaged or opened do not use the products. Check the expiry date and the gauge. Store in dry place, between 5 – 30 °C temperature, protect from sun light. Possible allergic reactions should be considered. It must be used only in hospitals by physicians who familiar with the device and procedures. For single Patient Use Only. Do not attempt to clean or resterilize this product. Do not use the device under MRI. After use, this product may be a potential biohazard. Handle in a manner which will prevent accidental puncture. Dispose in accordance with applicable laws and regulations.

Please double check for device function and possible damage for correct implantation and do not use if package is opened or damaged.

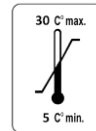
PROCEDURE:

1. Using the sterile technique, prepare the skin with antiseptic and drape.
2. Using Chiba Needle, infiltrate the selected area with local anaesthetic.
3. Attach the Needle Guide to the USG transducer
4. Insert the biopsy needle through Needle Guide
5. Put the biopsy samples to the bottles.

Note: These instructions are NOT meant to define or suggest any medical or surgical technique. The individual practitioner is responsible for the proper procedure and techniques to be used with this device.

Single Use

Storage limits

Expiry Date

STERILE EO

Ethylen oxide sterilized

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GEOTEK MEDİKAL VE SAĞLIK HİZ.LTD.STİ