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### INSTRUCTIONS FOR USE:

ENG

GEOTEK Percutaneous Access needle is a needle with depth marks, stopper and echogenic marker placed on the distal tip.

**Indications For Use:** Mainly used in interventional radiology and urology procedures, as a puncture needle, for percutaneous access.

**Contraindications:** Biopsy / aspiration sets or needles should be used by a physician trained in interventional techniques and familiar with the possible side effects, typical findings, limitations, indications and contraindications of biopsy procedures. Physician judgement is required when considering biopsy on patients with bleeding disorder, or receiving anti-coagulant medications.

**Possible Side Effects:** Pain, fever, bleeding, hematuria and infection.

**Caution:** Ethylen oxide sterilized. Non-pyrogenic. Check the package. if package damaged or opened do not use the products. Check the expiry date and the gauge. Store in dry place, between 5 – 30 °C temperature, protect from sun light. Possible allergic reactions should be considered. It must be used only in hospitals by physicians who familiar with the device and procedures. For single Patient Use Only. Do not attempt to clean or resterilize this product. Do not use the device under MRI. After use, this product may be a potential biohazard. Handle in a manner which will prevent accidental puncture. Dispose in accordance with applicable laws and regulations.

**Please double check for device function and possible damage for correct implantation and do not use if package is opened or damaged.**

1. Using the sterile technique, prepare the skin with antiseptic and drape.
2. If necessary, infiltrate the selected area with local anaesthetic.
3. Open the needle pouch and remove protective sheath from the needle.
4. Under USG or CT Insert the needle and advance into the targeted area.
5. Remove the stylet.
6. Insert the guide wire through the needle.
7. Check the wire position and remove the needle.

**Note:** These instructions are NOT meant to define or suggest any medical or surgical technique. The individual practitioner is responsible for the proper procedure and techniques to be used with this device.

1984 : CE mark and NB identification number